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Doing a few things at world class is better than doing a lot of things less well.

Sectra Imtec products

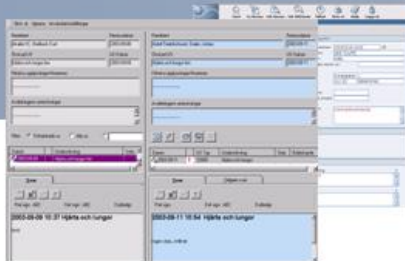
PD RADIT
Sectra PACS
Sectra RIS

PD Mammo
Sectra MicroDose
Mammography


PD MSA
Sectra Ortho package,
Osteoporosis package

...now there are more than...

...800 sites worldwide!

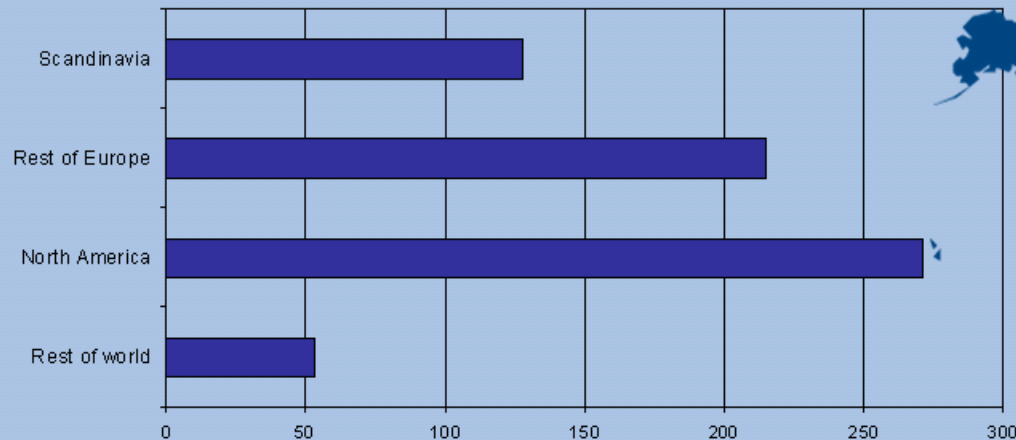Number of installations by geographic market

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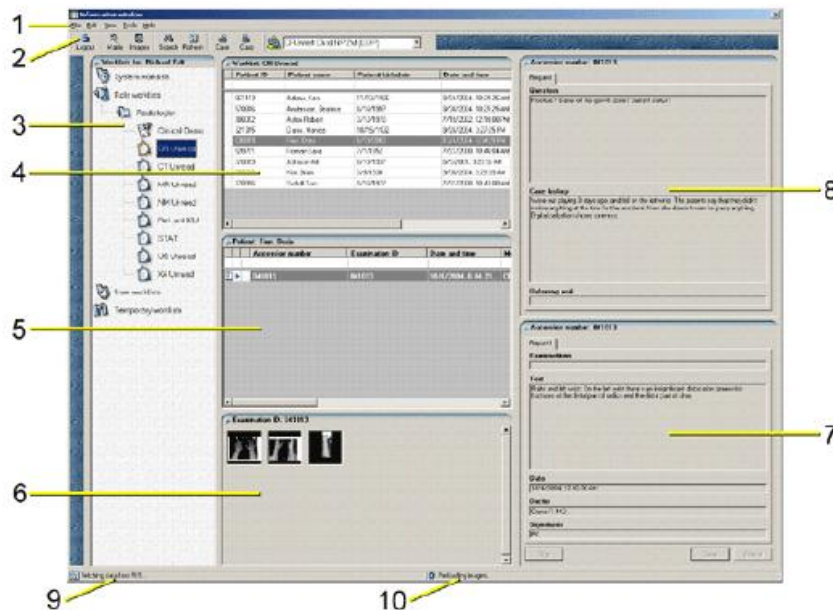
The Information Window consists of the following elements:

- Menubar (1) — Commands are available in drop-down menus on the menubar.
- Toolbar (2) — The configurable toolbar contains icon shortcuts to some of the commands available in the drop-down menus on the menubar.
- Worklist View (3) — Displays all available worklists.
- Patient List (4) — Displays all patients for the currently available worklist.
- Examination List (5) — Displays examinations for the currently selected patient.
- Series List (6) — The series of images for the currently selected examination are represented by thumbnails in the series list. Double-clicking on a thumbnail opens the Quick Window, displaying a full screen version of the thumbnail image.

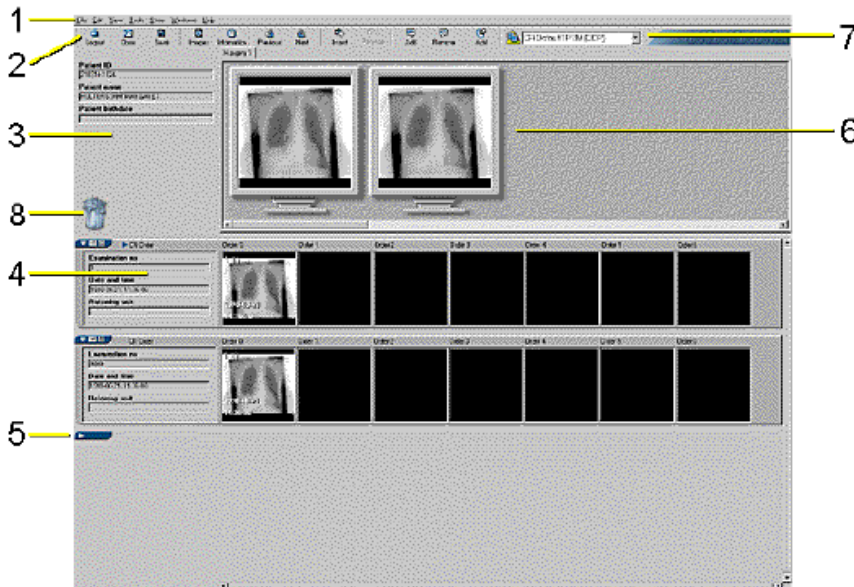
The Information Window.

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- Grensesnitt - Matrisevinduet

The Matrix Window consists of the following elements:


- Menubar (1) — Commands are available in drop-down menus on the menubar.
- Toolbar (2) — The configurable toolbar contains icon short cuts to some of the commands available in the drop-down menus on the menubar.
- Patient Text View (3) — This area shows the personal ID, name and date of birth for the current patient.
- Matrix (4) — Below the patient text view and the miniature screens is the matrix. The matrix consists of a number of rows, one for each examination. Each row begins with an examination text view, showing some data about each examination. Series icons are located to the right of the examination text view and each represents one series in the examination. At the bottom and right side of the matrix there might be scrollbars to allow scrolling in the matrix, if some icons do not fit on the screen. Note: If examination wrapping is used, the series icons may be wrapped onto several rows within each examination. However all series icons within a single examination should be regarded as located on the same matrix row, even though they may be wrapped onto several physical rows. To learn more about configuring examination wrapping, please refer to [section 12.7 Configuring Windows Properties](#).
- Collapsed exams (5) — Examinations which are not included in the presentation (according to the DDP or manual presentation) are collapsed.
- Miniature screens (6) — The miniature screens symbolize the screens of the computer where the radiologist will view the images for this case.
- Presentation Selector (7) — This drop-down menu shows the currently selected DDP or manual presentation.
- Recycle bin and Reject bin (8) — The recycle bin is used for permanently deleting images. The reject bin is used for deleting images from this examination but allowing the rejected images to be analyzed at a later time.

The Matrix Window.

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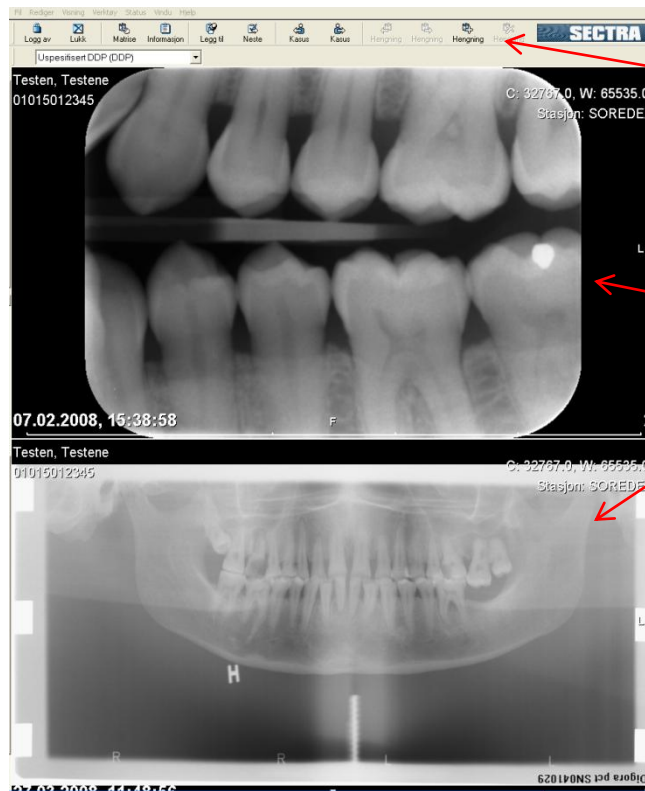- Grensesnitt - Bildevinduet

An Image Window consists of the following elements:

- Menubar (1) — Commands are available in drop-down menus on the menubar.
- Toolbar (2) — The configurable toolbar contains icon short-cuts to some of the commands available in the drop-down menus on the menubar.

The Image Window can be partitioned in different ways. Supported partitions are 1x1, 1x2, 1x3, 2x1, 2x2, 2x3, 3x1, 3x2 and 3x3. The 1x1 partition means that only one image appears on the screen. A 1x2 partition means that two images appear side by side on the screen, i.e. that the window comprises two *parts*. The figure above shows a 2x2 partitioning. It can be configured to toggle between the current partition mode and 1x1 mode by using a mouse double-click combination. Read more about this in [section 12.5 Configuring Image Tools Properties](#).

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Fødselsnummer	Pasientnavn	Fødselsdato	Date og tidspunkt	Modalitet	Kroppsdel
01015400000	Testa, Test	01.01.1954	06.03.2008, 10:39:47	IO	JAW
05067712345	Testen, Cat2	05.06.1977	07.03.2008, 10:00:00	IO	
02105054321	Testu, Testu	02.10.1950	06.03.2008, 09:57:17	IO	JAW
01020311111	Testy, Testya	01.02.1903	06.03.2008, 11:22:57	IO	JAW

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Fødselsnummer	Pasientnavn	Fødselsdato	Date og tidspunkt	Modalitet	Kroppsdel
3101016321	Test, Læbry	31.01.2001	05.03.2008, 10:11:40	IO	
01015400000	Testa, Test	01.01.1954	06.03.2008, 10:39:47	IO	JAW
01015012345	Testen, Testene	01.01.1959	03.03.2008, 11:47:50	IO	
05067712345	Testen, Cat2	05.06.1977	07.03.2008, 10:00:00	IO	
02105054321	Testu, Testu	02.10.1950	06.03.2008, 09:57:17	IO	JAW
01020311111	Testor, Testor	01.02.1903	06.03.2008, 11:22:57	IO	JAW

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Fødselsnummer	Fødselsdato	Date og tidspunkt	Modalitet	Kroppsdel
01015400000	01.01.1950	25.02.2008, 09:40:00	IO	
01015012345	01.01.1950	03.03.2008, 11:47:50	IO	

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